Xenoderm

GB Temporary biological skin replacement
Instructions for use

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In the event of a serious incident involving Xenoderm, please report it to MBP Medical Biomaterial Products GmbH and to the competent authority in your Member State.

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URL: <http://www.mbp-gmbh.de/xenoderm>

GB

Xenoderm - Temporary biological skin replacement

Definition

Xenoderm acts as a temporary skin replacement, within the limits of the functions of the patients own skin to:

- Decrease fluid and heat loss
- Enhance the growth of epithelial and granulation tissue
- Protect granulating wounds

Indication, Intended use

- Coverage of full-thickness or partial thickness skin loss due to burns or other trauma
- Coverage of chronic non healing wounds, such as venous, diabetic or pressure ulcers
- Coverage of autograft
- Coverage of donor sites

Advantages

- Simple application after reconstitution of Xenoderm
- Good adherence of the biological dressing
- Decrease in pain; exposed nerve endings covered
- No evidence of vascularization within seven days
- No sensitization by the patient to the dressing
- Protection of the underlying tissue against bacterial ingress
- Protection against the drying effects of the environment
- Reapplications are almost painless and without blood loss

- Transparency of the dressing allows inspection of the covered area
- Xenoderm facilitates movement, early ambulation and rehabilitation

Application

1. To reconstitute; the inner packet should be opened aseptically, and the dressing submerged in a basin containing sterile isotonic saline solution, at room temperature, for at least 2 minutes, or until the dressing becomes translucent.
2. At the time of initial application or re-application of Xenoderm, the wound area should be thoroughly debrided and gently cleansed, e.g. with an antiseptic; On partial thickness wounds Xenoderm may be left in place, once adherent, and allowed to slough off as re-epithelialization occurs.
3. Apply Xenoderm firmly to the wound, smoothing to eliminate bubbles and trimming to conform to the wound boundary.
4. As a precaution Xenoderm should not be applied to weight bearing areas of the body (risk of maceration).
5. Xenoderm can be held in place by gauze bandages or any other suitable dressing.
6. Xenoderm should be changed according to the dressing protocol of the unit.
7. Xenoderm dressing changes are almost painless. Always moisten the dressing before removal with sterile isotonic saline solution.
8. The length of time that Xenoderm should be left on the wound is variable and is dictated by the course of wound healing.
9. Infected, heavily secreting wounds may initially require more frequent dressing changes.

10. Xenoderm dressing changes on a daily basis may be required with the above wounds.
11. On all aseptic wounds –including granulating wounds- Xenoderm should be removed within 10 days due to the possibility of excessive adherence.

Contraindications

- Full-thickness burn wounds before removal of the eschar
- Badly contaminated wounds which have not been adequately debrided
- Patients with sensitivity to porcine material
- Xenoderm should not be used in combination with antiseptics that release chlorine. Protein-damaging substances (tannic acid, silver nitrate) should also not be used.

Use during pregnancy and lactation, use in children and elderly patients.

No studies are available on the use of Xe-noderm during pregnancy and lactation or on its influence on human reproductive capacity. Before using Xe-noderm, the attending physician must therefore weigh up the benefits for the

mother and the possible risks for the child on an individual basis.

There are no findings which suggest that special precautionary measures are necessary depending on the age of the patients to be treated.

Note

- Xenoderm is sterilized by γ -irradiation and must not be resterilized,
- is in a single sterile packet, must be considered nonsterile if the packaging is damaged and must be discarded,
- has a shelf life of five (5) years when stored at room temperature,
- must not be used after the expiry date,
- must be protected against moisture,
- is a medical device and therefore must be kept away from children.

After it has been removed from the packaging and/or has come in contact with a patient, Xenoderm surgical membrane cannot be reused without increased risk of contamination and subsequent infection.

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