MB-Collagen

GB Collagen dressing for wound care

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Sterile porcine collagen matrix

Effect

The porous structure of the collagen sponge produces a capillary suction effect and absorbs the discharge from the wound. The discharge from the wound dissolves the porous structure and releases the native collagen. This accelerates the building of new granulation tissue in the granulation phase. The afferent collagen has an indirect influencing control on the epithelization and regeneration phases by stimulating the body's own collagen synthesis, accelerating the epithelization process considerably. The quality of the newly formed tissue shows an increased capacity for improved cosmetic results.

Range of application

- Non-infected wounds with difficulties in healing
- Wounds with secondary wound healing in the granulation phase
- Wounds in the epithelization phase (e.g. ulcera cruris, decubitalulcera, split skin, burns, secondary healing wounds etc.)

Contraindication

Application on infected wounds

Side-effects

Allergic reactions may occur in isolated cases. Sometimes pain appears after application of a dry preparation to the wound surface. The increase of existing infection is very rare.

Use during pregnancy and breastfeeding

There has neither been any testing on the use of MB-COLLAGEN during pregnancy and breastfeeding nor on its impact on human

GB

MB-Collagen

Collagen dressing for wound care

fertility. Therefore the advantages for the mother and the possible risks for the child must be assessed individually before using MB-COLLAGEN.

Reciprocal effects

- MB-COLLAGEN should not be used in conjunction with antiseptics that release chlorine.
- Neither should substances that have a detrimental effect on protein (tannic acid, silver nitrate) be used.
- Wound disinfectants should not be used in combination with MB-COLLAGEN damp wound dressing.

Dosage and mode of application

Unless otherwise instructed by your doctor, enough collagen sponge should be placed on the dampened wound under aseptic conditions to cover the size of the wound. The wound should be fixed with a non-woven compress or lint compress. The rate of changing the dressing depends on the

intensity of the discharge from the wound:

- For intense discharge: several times a day
- For moderate discharge: once a day
- For slight exudation: after several days (provided that there are no signs of inflammation)
- If the exudation is very slight or there is no exudation, MB-COLLAGEN should be dampened with physiological sodium chloride or with Ringer solution.

Storage

The unopened, sterile packing must be stored under 24°C and under dry conditions with good ventilation. MB-COLLAGEN may not arrive into proximity of extreme temperatures and extreme humidity. MB-COLLAGEN may only be used before the date of expiry.

GB

MB-Collagen

Collagen dressing for wound care

Note

MB-COLLAGEN is sterilized by γ -irradiation and may not be resterilized. Sterility can only be guaranteed, if the packaging is intact.

After it has been removed from the packaging and/or has come in contact with a patient, the MB-COLLAGEN fleece cannot be reused without increased risk of contamination and subsequent infection.

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