# Diacoll

**GB** Hemostatic agent for topical use

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Sterile porcine collagen matrix

## Interaction with other agents

Antiseptics, which release chlorine (e.g. chloramine) as well tannic acid and caustics which change the protein must not be used together with collagen. Ointments, powder as well silicon preparations can change intercostal space at collagenfoil, -sponge and -fleece, and should not be used together with collagen.

## Indication areas

DIACOLL is indicated wherever capillary, venous, small arterial and diffuse seeping bleedings must be stopped, and where conventional means of hemorrhage control are either inadequate or technically difficult and time-consuming.

## Indications

General Surgery

Vascular, Cardiothoracic Surgery

Neurosurgery

Maxillofacial Surgery

ENT

Urology and Gynecology

## Dosage and application

After opening the package, DIACOLL is taken out with dry sterile instruments, placed on the cleaned sterile wound and pressed on lightly with appropriate resources (e.g. swabs). DIACOLL can be cut to the necessary size with sterile scissors. Immediately after contact with the wet wound surface DIACOLL sticks to the wound and forms a gel like bond with blood. Unnecessary collagen material should be removed. With superficial wounds an additional layer of cotton gauze fixed with elastic bandages is recommended.

## Contraindications

DIACOLL is contra-indicated in infected wounds, and must not be used concurrently with methyl methacrylate adhesive; it must not be used for intravascular application.

Hemostatic agent for topical use

---

If you use an absorbable hemostatic agent on or near bony or neural spaces:

- use the minimum amount necessary to achieve hemostasis; and,
- remove as much of the agent as possible after hemostasis is achieved.

This will reduce the likelihood of neural and other soft tissue damage from swelling of the absorbable hemostatic agent, and/or migration and swelling of fragments of the agent.

## Side effects

Allergic reactions may occur in isolated cases. Sometimes pain appears after application of a dry preparation to the wound surface. The increase of existing infection is very rare.

## Use during pregnancy and breast-feeding

There has neither been any testing on the use of DIACOLL during pregnancy and breastfeeding nor

on its impact on human fertility. Therefore the advantages for the mother and the possible risks for the child must be assessed individually before using DIACOLL.

## Note!

DIACOLL

- is sterilized by  $\gamma$ -irradiation and must not be resterilized
  - must be considered unsterile and must be discarded if the package is damaged
  - must not be used after the expiry date
  - must not be stored above 24°C
  - must be protected against moisture
  - is a medical device and therefore must be kept from children!
- After it has been removed from the packaging and/or has come in contact with a patient, the DIACOLL collagen fleece cannot be reused without increased risk of contamination and subsequent infection.

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